



CARE INSTRUCTIONS FOR CUBICLE CURTAINS

Wet cleaning: Use mild soap or detergent in warm water, not to exceed 160°F (synthetic setting). Use high water level with mild agitation. Extract gently. **DO NOT BLEACH.**

Drying: Air, cabinet or tumble dry at low temperature. Do not exceed 3 to 5 minutes on synthetic cycle at maximum of 140°F. Hang or fold immediately after drying.

Finishing: If necessary, press as follows:

- Hand iron at low heat setting (275°F) with damp press cloth or steam iron with dry press cloth.
- Grid Head Press - press for short intervals with minimum steam. Do not lock head.
- Flat Bed Press - press dampened drapery using cloth covering.

Do NOT press SNAPS, HOOKLESS RINGS or MESH.

AVOID PROLONGED CONTACT WITH HEAT.